AURA B • ALIA B • EUROPA B EVO

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At Tecno-Gaz S.p.A., sterilisation is our company mission, and that's why we've invested resources and time and energy over the last twenty years to developing **one of the most comprehensive sterilisation projects worldwide**. We work on a daily basis to give our clients new solutions and expertise to **help them improve their operations and make sure sterilisation is a safe process**.

SETTING UP A STERILISATION SYSTEM

As far as safety at a health practice is concerned, sterilisation is the first aspect to consider. Setting up a sterilisation system is important, as the operations and safety of a practice are based on this. **Autoclaves are devices offering physical and legal safety. They provide a service of vital importance, and this is why autoclaves must be chosen according to rational logics and based on professional and not only cost-related evaluations.** Choosing a leading product means using a device which has already been adopted by thousands of professionals the world over and this is without a doubt the best reference. **Tecno-Gaz S.p.A.** has been a leader and key player on an international level for years in the field of sterilisation and prevention too.

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Autoclaves and loads to sterilise

With the new EN 13060 standard, choosing the right autoclave has been made simple for professionals. The standard specifically indicates load classification, as explained below:

So based on the type of materials to sterilise, each professional must choose a **certified, adequate model**.

A dental surgeon must obviously choose an autoclave that can sterilise B type loads, as hollow materials such as handpieces and turbines have to be sterilised.

Tecno-Gaz has made history in this segment, with its line of Europa autoclaves and it now produces the Europa B Evo, Alia B, Aura B, autoclave, for outstanding sterilisation of B type loads.

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Hollow bodies

You can sterilise hollow bodies and porous objects with a guarantee of absolute quality. The Europa B Evo also lets you sterilise handpieces and turbines with the utmost safety.

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PID system

The main commitment of a company like Tecno-Gaz is to manufacture products with unique features, distinguishing traits and concepts that ensure maximum reliability and safety to the user.

Our research offices, together with support and assistance of external institutions have devised an extraordinary system that has been applied for the first time to our autoclaves. This innovative device is called P.I.D.

P.I.D. means Proportional Integral Derivate and represents the apex of closed loop controls. The structure of this technologically advanced control is simple and flexible. P.I.D. ensures a constant control and monitoring of the whole operating system of the autoclave, acting automatically in case of anomalies or incorrect values in technical and electronic parameters. This device behaves like a "real technician" with real-time diagnostic and intervention capability. This solution reduces anomalies and warnings, granting maximum reliability and minimising the machine inactivity times (if any).

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Loading water tanks: set the autoclaves according to your needs

You can choose how to pre-set Your autoclave.

Working autonomously, using water in the inner tanks and operating loading and manual unloading of pure and used water.

Direct connection to the water supply network by using the Deioniser (**Pura** model), thus allowing You to have automatic water supply and discharge without losing any time in manual operations.

Internal tanks

All the autoclaves are equipped with an internal tank for pure water and another for used water. Each tank is equipped with probes to detect the minimum and maximum water levels.

Tank filling

If the autoclave is connected with the **Pura** device the tanks are automatically filled, but if you want to use it autonomously loading shall be carried out manually. Tecno-Gaz autoclaves, to facilitate filling operation, are equipped with a pump-operated filling system featuring a level sensor that stops water filling if the tank is full. This useful option is included on all the models of the line. It avoids the risk of human errors due to excess pouring and it also reduces operator efforts. The autoclave is preset to allow manual filling of the tank in case the pump is out of order.

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Features

Preset temperatures

The preset temperatures used in our autoclaves' cycles are 134°C and 121°C. 134°C cycles are normally used for sterilization of solid and ferrous materials. 121°C cycles are normally used for sterilization of thermoplastic and delicate materials. **In sterilizing materials, always follow the recommendations of the device manufacturer.**

Ease of use

Choosing and selecting a sterilization cycle on Tecno-Gaz autoclaves is extremely simple and intuitive. One of the buttons makes it possible to scroll and view data about all sterilization cycles on the display, thus allowing You to find the cycle that suits your materials best, then You will only have to push another button to automatically start the whole sterilization cycle without having to worry about anything. In case of any error or warning a series of messages and LEDS will attract your attention to the problem.

Fully automatic cycles

All cycles are fully automatic, with all steps managed by the leading-edge computer board with a powerful microprocessor. Each step (vacuum, preheating, exposure time, drying) is automatically monitored, assisted and handled. There is no need for human intervention in cycle management.

Cycles without operator intervention and with auto-shutdown

All sterilization cycles have an automatic function that, if the operator is not by the machine when the cycle ends, will trigger an anti-condensation process and auto-shutdown operation. This system makes it possible to economize and increase safety of the device.

Test cycles

Our autoclaves are equipped with test cycles for periodic control of the device by means of physical and functional tests.

Helix test, to assess vacuum capacity in hollow bodies. **Bowie & Dick test** to evaluate vacuum capability in hollow bodies.

Vacuum test to evaluate vacuum retaining capability.

Constantly monitored cycles

The autoclaves **Tecno-Gaz** are equipped with a device used to constantly control and monitor the sterilization cycles. Any anomaly or uncommon feature is quickly detected and, if it jeopardises the sterilization cycles, immediately blocked. The operator is then warned of the anomaly by a message.

All cycles ensure sterilization of b-type loads

All the sterilization cycles feature the fractional vacuum system as well as controls and parameters that ensure sterilization of B-type loads. Therefore, even if the type of cycle chosen is incorrect the sterilization carried out on the material will be equally adequate.

Ergonomic structure and ease of use

The sterilization cycle is chosen by pushing one button and activated by pushing another button.

It is impossible to make a mistake or to choose a cycle that provides inadequate sterilization, all operational errors are notified by error messages.

The autoclave can be used by more than one person without risking to cause problems or damages.

Tecno-Gaz wanted to provide maximum operation safety even on this important issue.

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The AURA project

The Aura project has the aim to develop a new concept of sterilization, not only a new autoclave. The result is the use of the most advanced technology together with the unique experience of an international market leader.

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Why choosing AURA B

AURA B is representing a revolution in the world of the sterilizers designed for the B type loads. It is the first autoclave in the world which is able to connect to Internet, for the software upgrade, to upload the sterilization data, to get technical support in real time and to receive useful information directly on the display of the device. AURA B is provided with a remote control, which has the function to pilot some basic functions and which communicates with the equipment, to inform the operator about the end of the cycles. Useful voice alert supports and informs the operator in different procedures, by making the interact with the autoclave easy and intuitive.

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The first autoclave
connected to internet!

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Internet connection **for remote assistance**
for sending information to the customer
for gathered data saving

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AUTOMATIC UPGRADES

The autoclave looks for technical updating available on Tecno-Gaz server by downloading them directly through internet connection. This solution allows a continuous update and makes the autoclave a technically leading product.

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INFORMATICS AT YOUR SERVICE

By connecting the autoclave to a PC, it will be possible to be connected to a special portal, where several services can be found: data recording, management of equipments, user's manuals download, certificates download.

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INTERACTIVE DISPLAY

For the first time ever you will have an office device which can become also a communication tool. On the AURA display indeed, you can receive messages, invitations, offers from Tecno-Gaz or from the retailer who sold you the device. An excellent system to have new opportunities and information all the time.

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SOS SERVICE

Service of call via the internet, that if activated sends a message to the after sales operation headquarter, that will immediately contact the structure. This service will enter a project of global assistance, that will have the target to give the maximum support to all Tecno-Gaz customers.

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TRACEABILITY LABEL PRINT SYSTEM FOR TRACEABILITY

By using a relevant software it is possible to print special labels with or without bar code, that are to be stuck on the sterilization bag for correct instrument traceability.

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SHOPPING SERVICE

Thanks to the internet connection through general software putting in a direct order for all autoclave spare parts or accessories becomes possible.

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P.I.D. SYSTEM

P.I.D. ensures a constant control and monitoring of the whole operating system of the autoclave, acting automatically in case of anomalies or incorrect values in technical and electronic parameters.

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AUTOMATIC LOCK SYSTEM

The locking system is based on an electric-electronic and dynamic mechanism managed by the vacuum pump. This allows a perfect locking even if the gasket is old or worn. This solution relies on several safety systems and on a continuous locking of the door.

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BIOLOGICAL INCUBATOR

Periodically check the autoclave working order is a duty set forth by the current regulations. Tecno-Gaz has thus equipped AURA B autoclave with a system for the connection to and the management of the biological incubator.

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FITTING FOR PURA DEIONISER

The **Pura** osmosis system is an extremely compact, osmosis device designed for connection directly to the autoclave, to guarantee a constant supply.

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EVERYTHING WITHIN REACH

The autoclave is supplied with a remote control that can be used for:

- selection of cycles and starting of autoclave at distance
- visual control at distance of the performance and the status of the autoclave
- visual signal with vibration to indicate end of the cycle or error

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LISTEN TO YOUR AUTOCLAVE!

By sending out vocal signals it guides the operator on its usage, informing about anomalies and errors and leading to their solution. An exclusive option that assures saving of time, immediacy and maximum support to the operator;

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Why choosing ALIA B

ALIA B is a last generation technology autoclave, deriving from the innovative AURA project. This product results from a deep technological evolution, achieved with a long research and development activity.

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Constant control and monitoring of the entire autoclave functional system.

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ALIA B adopts a new cooling system, with double-body radiator.

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Sterilization Cycles 121°C-134°C suitable for both enveloped and free materials. All cycles are for **type-B loads**.

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ALIA B is managed by a new electronic system, equipped with **last-generation microprocessor**.

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SD CARD | THE MEMORY CARD

Collection and registration of data related to cycles and monitoring of the autoclave are carried out through SD card.

BIOLOGICAL INCUBATOR

Periodically check the autoclave working order is a duty set forth by the current regulations. Tecno-Gaz has therefore equipped ALIA B autoclave with a system for the connection to and management of the biological incubator.

AUTOMATIC LOCK SYSTEM

It includes a telemetric control system which monitors the autoclave and intervenes when there is an error. This is an important service which ensures the instrument's efficiency and protects the operator as required by law.

FITTING FOR PURA DEIONISER

The Pura osmosis system is an extremely compact, osmosis device designed for connection directly to the autoclave, to guarantee a constant supply.

CICLI / CYCLE	Tempo medio ciclo complessivo* / Average time of the total cycle*					
121° imbustato / 121° wrapped	60 - 65 min	60 - 65 min	60 - 70 min	60 - 70 min	60 - 70 min	56 - 66 min
134° flash / 134° flash	31 - 36 min	31 - 36 min	31 - 41 min	31 - 41 min	31 - 41 min	31 - 41 min
134° imbustato / 134° wrapped	47 - 52 min	47 - 52 min	47 - 57 min	47 - 57 min	47 - 57 min	43 - 53 min
134° Prion / 134° Prion	62 - 67 min	62 - 67 min	62 - 72 min	62 - 72 min	62 - 72 min	58 - 68 min

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* Times expressed in minutes (max with cold machine, min. with warm machine). The total time includes: vacuum, pre-heating time, cycle and drying phase. The times are to be intended for an autoclave with full loading.

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Why choosing Europa B Evo

Europa B Evo is a last generation autoclave, developed with the collaboration of the research office Tecno-Gaz and with other external important structures of development.

Europa B Evo is essential in all medical and dental structures and in every field where it is necessary to have safe sterilization for all materials: hollow, porous, solid, free and enveloped. A necessary device in order to assure the utmost preservation for operator and patient and an absolute operative serenity. **Europa B Evo** is the continuity of a success that Tecno-Gaz is obtaining for many years, and, in fact, thousand of professional people use Tecno-Gaz autoclaves with great satisfaction.

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Safety on the door

We have applied three safety devices on the autoclave door:

- **The first device is a mechanic and electro-magnetic one:** it triggers when the sterilization cycle begins and completely blocks the door opening. This device can be removed at the end of the cycle by pushing the device-unlocking button, but only if positive pressure is near zero.
- **The second device is an electrical one:** it triggers automatically when an attempt to force the door open is detected.
- **The third device is dynamic** and acts on the structure of the gasket.

Safety devices in place

- Certified safety valve
- Ped on sterilization chambers.
- Insulation of the chamber with certified ceramic material.

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Close-up view of the front of an Europa B Evo autoclave

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The Pura osmosis system is an extremely compact, osmosis device **designed for connection directly to the autoclave**, to guarantee a constant supply. The system can be installed in a number of positions to facilitate mounting and the optimum organisation of the available space. **Pura is fitted with a series of osmotic membranes and filters**, with an average life of between 600 and 800 cycles, depending on the original hardness of the water. The cartridge should therefore need replacing about once a year, or after 800 cycles. **Changing the filter is an extremely simple process** and can easily be carried out by the users. **The overall cost of the filters is very reasonable**, and the cost per litre is therefore almost negligible. When the **Pura system** is not producing water, it is not pressurized, and this avoids all kinds of problems caused by the accidental breakage of couplings and fittings.

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All rights reserved. Variations can be done without notice. Tecno-Gaz S.p.A. is to be considered not responsible for damages caused by the lack or the wrongness of the information here mentioned.